OZONE STRIP™ Chemical Indicator

Product description

The OZONE STRIP™ Chemical Indicator is a paper strip 2 cm wide by 5,5 cm long fabricated with a chemical indicator paper that turns from blue to white when exposed to the ozone sterilization process. The REF color represents the color limit the indicator has to reach; the maximum color limit being white.

The OZONE STRIP™ Chemical Indicator is designed to indicate whether appropriate conditions in the SteriBox™ V-Series were reached for successful ozone sterilization. The OZONE STRIP™ Chemical Indicator is a type 4 chemical indicator as categorized by ISO 11140-1:2014. The indicators respond to ozone, humidity, temperature and exposure time.

Indications for Use

The OZONE STRIP™ Chemical Indicator is designed to monitor all critical parameters for ozone sterilization. Only use in accordance with the conditions of use described in this document.

Contraindications

Do not use OZONE STRIP™ Chemical Indicator to monitor steam, dry heat, ethylene oxide, hydrogen peroxide or any other sterilization processes.

Precautions


Make sure to place the OZONE STRIP™ Chemical Indicator as indicated in the Directions for Use.

Use only within temperature range min. 20°C max. 30°C.

Directions for Use

Unwrapped instruments:

Take one OZONE STRIP™ Chemical Indicator out of the pack and place it as shown.

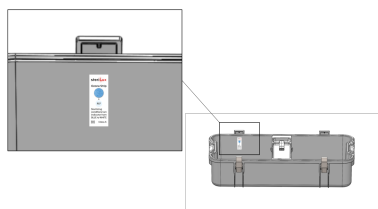Wrapped instruments:

Place one OZONE STRIP™ Chemical Indicator inside each pouch. Make sure the chemical indicator is facing the paper part of the pouch.

After processing, the chemical indicator will be the same color or lighter color than the REF color shown.

Storage

- Store in a dry (30 – 60% relative humidity) condition at room temperature (15 – 30°C), and protected from light.
- After use, the indicator will not change visually within 6 months when stored at recommended conditions.

Explanation of Symbols

	Catalogue Number
	Batch Number
	Date of manufacture
	Use-by date
	Do not reuse
	Caution, see instructions for use
	Humidity limitation
	Temperature limit
	Keep dry
	Keep away from sunlight
	Non-sterile
	Recyclable
	Manufacturer

Made in Switzerland by
SteriLux SA
Chemin du Viaduc 12
1008 Prilly, Switzerland

© 2019, SteriLux SA.
All rights reserved.
Issue Date: 2019-10

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