HUMIDIFY™ Blotting Paper

Product description

The HUMIDIFY™ Blotting Paper is a white paper strip 6 cm wide by 16 cm long weighting 300 g/m². It is designed to bring the ideal humidity level in the SteriBox™ for the sterilization process. One HUMIDIFY™ Blotting Paper has to be used for each sterilization cycle to ensure optimal conditions for sterilization success.

Indications for Use

The HUMIDIFY™ Blotting Paper strip is designed to release the appropriate amount of humidity in the SteriBox™ for ozone sterilization.

Contraindications

None.

Precautions

Do not place the HUMIDIFY™ Blotting Paper in direct contact with instruments. Make sure to place the amount of water recommended on the HUMIDIFY™ Blotting Paper.

Directions for Use

1. Remove the module from the SteriBox™
2. Place one HUMIDIFY™ Blotting Paper on the module
3. Add 5 mL of distilled water at 60°C on the HUMIDIFY™ Blotting Paper using a liquid dispenser; a syringe and a kettle with adjustable temperature control are recommended to heat up the water and dispose the appropriate amount of water.
4. Replace the module inside the SteriBox™

Storage

- Store in a dry (20 – 70% relative humidity) condition at room temperature (5 – 40°C), and protected from light
- After use, dispose of the HUMIDIFY™ Blotting Paper in recycling bin

Explanation of Symbols

	Catalogue Number
	Batch Number
	Date of manufacture
	Use-by date
	Do not reuse
	Caution, see instructions for use
	Humidity limitation
	Temperature limit
	Keep dry
	Keep away from sunlight
	Non-sterile
	Recyclable
	Manufacturer

Made in France for
SteriLux SA
 Chemin du Viaduc 12
 1008 Prilly, Switzerland

© 2019, SteriLux SA.
 All rights reserved.
 Issue Date: 2019-07

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