HUMIDIFY™ Blotting Paper

Product description

The HUMIDIFY™ Blotting Paper is a white paper strip 6 cm wide by 16 cm long weighting 300 g/m². It is designed to bring the ideal humidity level in the SteriBox™ for the sterilization process. One HUMIDIFY™ Blotting Paper has to be used for each sterilization cycle to ensure optimal conditions for sterilization success.

Indications for Use

The HUMIDIFY™ Blotting Paper strip is designed to release the appropriate amount of humidity in the SteriBox™ for ozone sterilization.

Contraindications

None.

Precautions

Do not place the HUMIDIFY™ Blotting Paper in direct contact with instruments. Make sure to place the amount of water recommended on the HUMIDIFY™ Blotting Paper.

Directions for Use

1. Remove the module from the SteriBox™
2. Place one HUMIDIFY™ Blotting Paper on the module
3. Add 5 mL of distilled water on the HUMIDIFY™ Blotting Paper using a liquid dispenser; a syringe is recommended to dispose the appropriate amount of water.
4. Replace the module inside the SteriBox™

Storage

- Store in a dry (20 – 70% relative humidity) condition at room temperature (5 – 40°C), and protected from light
- After use, dispose of the HUMIDIFY™ Blotting Paper in recycling bin

Explanation of Symbols

- Catalogue Number
- Batch Number
- Date of manufacture
- Use-by date
- Do not reuse
- Caution, see instructions for use
- Humidity limitation
- Temperature limit
- Keep dry
- Keep away from sunlight
- Non-sterile
- Recyclable
- Manufacturer

Made in France for
 SteriLux SA
Chemin du Viaduc 12
1008 Prilly, Switzerland

© 2019, SteriLux SA.
All rights reserved.
Issue Date: 2019-07

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