OZONE STRIP

Type 1 Chemical Indicator

Product description

The OZONE STRIP Type 1 Chemical Indicator has a round-shaped indicator ink printed on a white paper strip. The chemical indicator ink turns from purple to green after exposure to ozone.

Indications for Use

Use the OZONE STRIP Type 1 Chemical Indicator as an external pack process indicator to exposure to ozone in the SterOx System V-Series. The OZONE STRIP Type 1 Chemical Indicator turns from purple to green after exposure to ozone.

Contraindications

None.

Precautions

1. Do not use the OZONE STRIP Type 1 Chemical Indicator in sterilization processes which it is not designed to monitor, such as:
 - Steam sterilization cycles,
 - Dry heat sterilization cycles,
 - Ethylene oxide sterilization cycles,
 - Hydrogen peroxide sterilization cycles or
 - Other chemical vapor or low temperature sterilization processes.
2. To prevent color change of the indicator ink before use, do not store near sterilants or other chemicals.
3. Do not place indicator strip in direct contact with instruments. Under certain sterilization conditions, staining/transfer of ink may occur. Do not use contaminated instruments.

Directions for Use

1. Carefully tear or use scissors at the tear notch at the top of the foil package to make the initial opening. Remove only the number of OZONE STRIP Type 1 Chemical Indicator needed. Reseal the package.
2. Before use, confirm that the unprocessed chemical indicator is purple. If it is not purple, do not use.
3. Place the OZONE STRIP Type 1 Chemical Indicator on each peel pouch or pack to be processed.
4. Process the load according to the SterOx System V-Series instructions.
5. After processing, the indicator ink becomes green. If the color change is not complete, inadequate exposure to ozone is indicated. The pack should be returned for reprocessing.

Storage

- Unopened and resealed packages of OZONE STRIP Type 1 Chemical Indicator are best stored at normal room temperature (15 - 25°C) and relative humidity < 50%.
- After use, the indicator will not change visually within 4 weeks when stored at above conditions.

Explanation of Symbols

	Catalogue Number
	Batch Number
	Date of manufacture
	Manufacturer
	Consult instructions for use
	Use-by date
	Do not reuse
	Humidity limitation
	Temperature limit
	Keep dry
	Keep away from sunlight

Made in Japan for

 **SteriLux SA**
Chemin du Viaduc 12
1008 Prilly, Switzerland

© 2021, SteriLux SA.
All rights reserved.
Issue Date: 2021-02

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