OZONE STRIP

Type 4 Chemical Indicator

Product description

The OZONE STRIP Type 4 Chemical Indicator is a paper strip 2 cm wide by 5,5 cm long fabricated with a chemical indicator crepe paper that turns from blue to white when exposed to the ozone sterilization process. The REF color represents the color limit the indicator has to reach.

Indications for Use

The OZONE STRIP Type 4 Chemical Indicator is designed to indicate whether appropriate conditions in the SteriBox V-Series were reached for successful ozone sterilization. It is a multi-variable indicator as categorized by ISO 11140-1:2014. The indicators respond to ozone, humidity, temperature and exposure time. The OZONE STRIP Type 4 Chemical Indicator turns from blue to REF after exposure to a successful cycle in the SterOx System V-Series.

Contraindications

None.

Precautions

- Do not use the OZONE STRIP Type 4 Chemical indicator in sterilization processes which it is not designed to monitor, such as:
 - Steam sterilization cycles,
 - Dry heat sterilization cycles,
 - Ethylene oxide sterilization cycles,
 - Hydrogen peroxide sterilization cycles or
 - Other chemical vapor or low temperature sterilization processes.
- To prevent color change of the indicator before use, do not store near sterilants or other chemicals.
- Do not place indicator crepe paper against any surface as it may prevent color change.
- Do not use in Speed cycles.
- Use only within operating temperature range min. 20°C max. 30°C.

Directions for Use

- Open package and remove one OZONE STRIP Type 4 Chemical Indicator. Reseal the package.
- Before use, confirm that the unprocessed chemical indicator is blue. If it is not blue, do not use.
- Place the OZONE STRIP Type 4 Chemical Indicator as shown on the scheme (below the latch).
- Process the load according to the SterOx System V-Series instructions.
- After processing, the indicator has turned to the REF color or lighter. If the color change is not complete, inadequate exposure to ozone or too low relative humidity is indicated. The content should be returned for reprocessing.

Storage

- Unopened and resealed packages of OZONE STRIP Type 4 chemical Indicator are best stored at normal room temperature (15 - 25 °C) and relative humidity < 50%.
- After use, the indicator will not change visually within 4 weeks when stored at above conditions.


Explanation of Symbols

- | | |
|---|-----------------------------------|
|  | Catalogue Number |
|  | Batch Number |
|  | Date of manufacture |
|  | Manufacturer |
|  | Caution, see instructions for use |
|  | Use-by date |
|  | Do not reuse |
|  | Humidity limitation |
|  | Temperature limit |
|  | Keep dry |
|  | Keep away from sunlight |

Made in Switzerland by

 **SteriLux SA**
Chemin du Viaduc 12
1008 Prilly, Switzerland

© 2021, SteriLux SA.
All rights reserved.
Issue Date: 2021-02

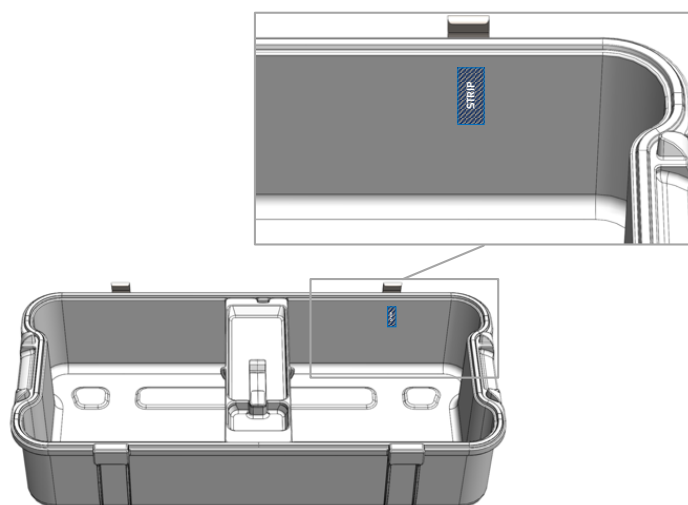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